


Using KATANA™ Cleaner and PANAVIA™ SA Cement Universal on a Lithium Disilicate Crown

Case and Images Courtesy of Richard Young DDS, San Bernardino, CA



HF Etch & Try-In Complete

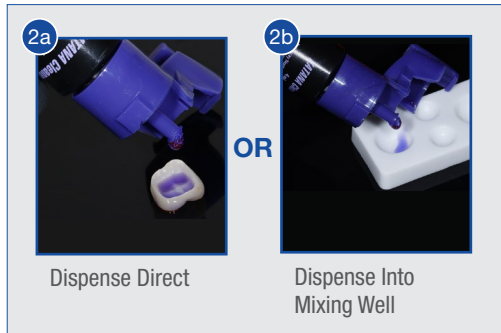

Dispense Direct

Dispense Into Mixing Well



Rub 10 Seconds, Rinse & Dry



Rub 10 Seconds Rinse & Dry

MDP-Based Surfactant Breaks Down Blood, Saliva etc....



Inject Cement Directly Into Any Glass Ceramic, Metal, Zirconia or Composite Restoration



Fresh Activated Silane With Every Mix



Tack-Cure 2-5 Seconds



Curing Chemistry Provides Nice Gel-State For Easy Removal



Final Restoration